mediCipio[®] A

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Instructions for Use

Product description:

mediCipio® A is a sterile, fully degradable pad consisting of oxidized regenerated cellulose (ORC) and collagen. To facilitate visual distinction between the layers, the ORC component is coloured yellow with riboflavin. The combination of the layered structure and added glycerol creates a flexible pad.

Ingredients:

- oxidized regenerated cellulose
- riboflavin (colouring agent E101)
- collagen (porcine origin)
- glycerol

Areas of application:

mediCipio® A is a mechanical haemostyptic pad that is used, both invasively and non-invasively, as a local supporting measure to improve haemostasis where there is diffuse bleeding and capillary oozing; it can also be used as a tissue sealant.

Mechanism of action:

The mediCipio® A pad provides a matrix for facilitating blood clot formation. By absorbing liquid, the pad leads to the local concentration of coagulation factors and thrombocytes, which boosts blood clotting at the desired location. In addition, mediCipio® A creates a barrier to blood flow.

Usage information:

mediCipio® A is double packaged. Both packages are peelable. The outer clear-film package acts as the sterile barrier system. The inner package provides a light barrier that protects the yellow ORC layer from decolouration. Both packages must only be opened immediately before use. The product must be removed and applied under sterile conditions and is immediately ready for use. It is advisable to moisturize dry wounds with a suitable physiological rinse solution before application. Place the **yellow side** of mediCipio® A onto the wound and apply gentle pressure for three to five minutes. The number of pads that need to be applied will depend on the tissue to be treated. If multiple pads are used, they should be applied so that they overlap at the edges. The pads can be cut to size if required. The pads are best handled with dry gloves and surgical instruments. Do not use more material than is necessary. It is recommended to document the name of the patient and the product batch number with each application of mediCipio® A. mediCipio® A is designed to be left *in situ* but can be removed once it is no longer therapeutically useful. It is recommended that the pads are removed in critical areas, such as in the vicinity of nerves or sensitive tissue. ?

Contraindications:

mediCipio® A must not be used on patients with known hypersensitivity to at least one of the ingredients of the two-layer pad. Do not press mediCipio® A pads into blood vessels or apply intravascularly. The use of mediCipio® A is contraindicated in infected areas. Do not use mediCipioA to prevent tissue adhesions.

Warnings and precautionary measures:

Do not use the product if the packaging is damaged. Once the package has been opened, the product must not be kept or stored for later use. The product has been sterilized by irradiation. Do not resterilize. The product should be kept dry before use. Premature exposure to moisture will reduce the efficiency of the product. mediCipio® A is for single use only. If the product is reused, there may be a risk of infection and the product will not perform as intended. Adequate measures, such as cleaning the wound area, can prevent adhesions of the surrounding tissue. Caution is indicated in patients with coagulation disorders, as prolonged hemostasis and/or a less developed occlusive thrombus may occur. There is no experience of use in children, pregnant women and/or breastfeeding women. There is also no experience of use in the immediate vicinity of bones. Hence, such applications should only be made if in the individual risk-benefit assessment by the attending physician the expected benefit clearly outweighs the risk.

Interactions:

None known at present.

Adverse reactions:

None known at present.

Shelf life:

If stored properly, the product has a shelf life of five years.

Storage information:

Store in a dry place away from direct sunlight at a temperature below 25 °C.

Available sizes:

10 x 10 mm	1 x 5 cm
10 x 20 mm	2,5 x 3 cm
20 x 20 mm	5 x 5 cm
	5 x 10 cm