## Mode d'emploi

mediRegis® C

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## Instructions for use

mediRegis® C

Collagen pad for Dental and Oral Surgery Applications

### 1. Product Description

mediRegis® C is a sterile collagen fleece for use in dental and oral surgery. The collagen fleece is soft, flexible and completely resorbable. The primary material is collagen from porcine dermis.

### 2. Indication

mediRegis® C is indicated for wound care and to support healing and tissue regeneration in dental and oral surgery applications. The following applications are indicated:

- Support of haemostyptic wound care of the extraction socket
- Reduction of secondary bleedings
- Reduction of the risk of wound infections after tooth extraction
- Initial regeneration of the extraction socket (socket preservation)
- Protection of the Schneiderian membrane from perforation
- Care of oral wounds
- Care of soft tissue defects
- Promotion of wound healing and support of tissue regeneration

### 3. Contraindications

mediRegis® C is contraindicated in patients with known sensitivity against porcine collagen. mediRegis® C is contraindicated in area of infection.

### 4. Warnings, Precautions

The product is designed for single use. Reuse is a potential risk of infection; in addition, efficacy can no longer be guaranteed with reuse.

mediRegis® C may not be resterilized.

Sterility is guaranteed as long as the individual packages have not been opened or damaged. Products from opened individual packages must not be stored for further use.

Do not use products after expiry date or from damaged individual packages.

### 5. Application

I. Thoroughly clean field of application. If necessary, remove inflamed tissue and disinfect with suitable rinsing solution.

II. Select suitable size of mediRegis® C and/or adapt to defect through cutting, rolling or folding.

III. Place mediRegis® C over/in defect site and adapt through moderate pressure. Duration of pressure exertion depends on the intensity of the bleeding. After a short time, the collagen fleece will be saturated with blood and will stabilize the wound.

IV. If necessary, suture the wound.

Since mediRegis® C is completely resorbable, it may remain in the wound until completion of the healing or regeneration.

### 6. Side Effects

Not known so far.

### 7. Durability

If stored properly, the shelf life is 5 years.

### 8. Storage Instructions

mediRegis® C is to be stored dry, protected from the direct sun light and at room temperature (do not store above 25° C).

### 9. Order Information

mediRegis® C is supplied in sterile, individual packages. The following sizes are available:

REF 011001	10 x 10 mm	10 pads per package
REF 011002	10 x 20 mm	
REF 011003	20 x 20 mm	

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