mediCipio®  - Collagen Pad

Gebrauchsinformation

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Directions for use

Product description:

mediCipio® C sterile wound pad is made up of pure collagen from porcine dermis.

Therapeutic indications:

mediCipio® C is used to cover wounds in the case of injuries involving extensive superficial and deeper tissue defects, e.g. burn wounds, surgical tissue defects, chronic wounds, ulcers of varying origin, e.g. venous ulcers or arterial ulcers, and wound cavities. If wound healing has so far stagnated with other wound treatments, sustained epithelisation can be initiated with mediCipio® C.

Note:

Thorough mechanical cleansing is required for wounds with a purulent discharge before mediCipio® C is applied.

Undesirable effects:

No known side-effects

Method of application and duration of use:

After disinfection of the skin and wound cleansing, lay the correct size of mediCipio® C onto the exuding wound. If the mediCipio® C wound pad does not fit the wound dimensions, it can be cut to the right size and shape under sterile conditions. If wound discharge is minimal or non-existent, it is advisable to moisten mediCipio® C with a suitable wound cleansing solution, such as Ringer's solution, after it has been placed in the wound. Gently pressing on the pad will improve contact with the surface of the tissue defect.

Change of pad or application of new wound dressing depends on the degradation or incorporation of the mediCipio® C wound collagen and epithelisation of the wound. If necessary, mediCipio® C can be gently washed out with a suitable wound cleansing solution, such as Ringer's solution, and removed. For fresh injuries, the dressing can be left on the wound until healing is achieved. Ulcers and heavily exuding wounds may make a daily change of dressing necessary. Otherwise the mediCipio® C wound pad can be left on the wound until healing is complete.

Note:

For single use only. The re-use of disposable products presents possible risks of infection for the patient and the user. Re-sterilization is not possible.

Shelf life and storage:

If properly stored, mediCipio® C is stable for 5 years. It should not be used after the expiry date. mediCipio® C should not be stored for further use once the sterile primary packaging has been opened. In case any damage is observed in the primary packaging, mediCipio® C must not be used. mediCipio® C must be stored in a dry and dark place at room temperature. Do not store above 25 °C.

Product Variants:

mediCipio® C wound pad is available in the following sizes:

Pack contents	Packungsinhalt	Packungsinhalt
2 pads	5 pads	10 pads
Dimensions	Dimensions	Dimensions
9,0 cm x 22,0 cm	2,2 cm x 2,7 cm	2,2 cm x 2,7 cm
19,5 cm x 22,0 cm	4,5 cm x 5,5 cm	4,5 cm x 5,5 cm
	4,5 cm x 11,0 cm	4,5 cm x 11,0 cm
	9,0 cm x 11,0 cm	9,0 cm x 11,0 cm

	
medichema GmbH Weststraße 57 09112 Chemnitz, Germany	medichema GmbH Weststraße 57 09112 Chemnitz, Allemagne
Information last updated: 2017 - 01	Version: 2017 - 01

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